



## Assigning an active tag to an animal

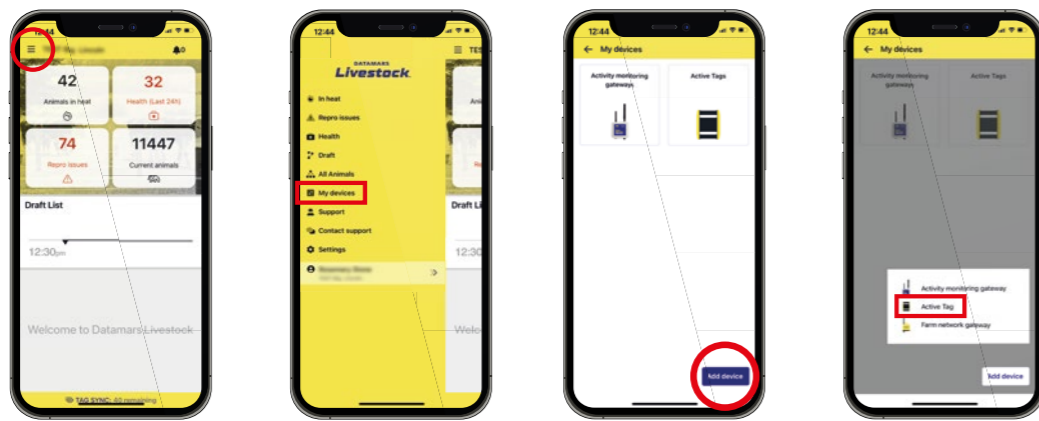


1. Go to > **My Devices** > **Add device** > **Active Tag** to begin pairing.
2. When prompted, swipe the magnet over the magnet icon until the LED flashes TWICE. Follow the instructions to pair the animal VID/LID or EID.

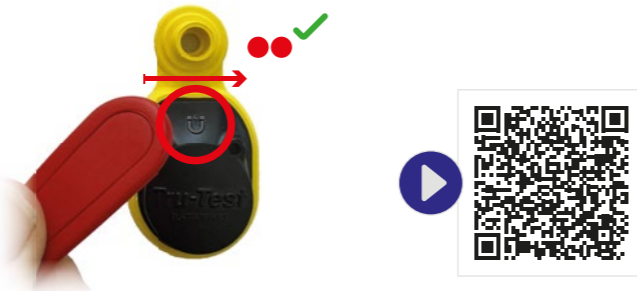


## Attaching an active ear tag to an animal

1. Insert the female part of the tag into the bottom of the Z Tags No-Tear-Tagger.
2. Push the male part of the tag onto the pin.

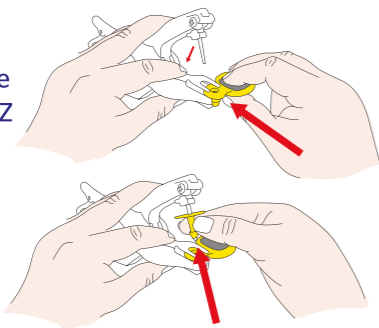

Do not attach the active ear tag too close to the animal's head or it will limit motion.

3. Attach the active ear tag to either ear, between the two large veins. The male part of the pin should be at the back of the ear.

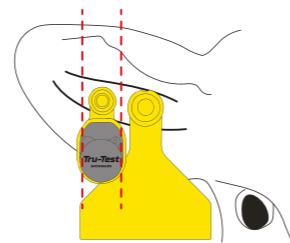

## Removing an active ear tag



1. Hold the animal's ear and the active tag in one hand.
2. Slide the Z Tags Multi-Cutter ear tag remover between the back of the ear and the tag and pull firmly to slice through the shaft of the ear tag.
3. Between animals, use animal-grade disinfectant to clean the cutter and the ear tag.

## Re-using an active tag on a new animal

Assign the active tag to a new animal in the usual way. It will automatically be unassigned from the old animal.

## Putting an active tag to sleep to conserve the battery

Hold the magnet over the magnet icon until the LED flashes ONCE.



## Marking an animal as 'sold' or 'died'

1. Log in to Datamars Livestock [monitoring.livestock.datamars.com](https://monitoring.livestock.datamars.com)
2. In the **Animals** screen, select an animal, click ... then **Mark as sold** or **died**

